

# Bionect®

(hyaluronic acid sodium salt, 0.2%)

BIONECT® Cream

BIONECT® Gel

BIONECT® Spray

For topical use only. Rx only.

**Product Description:** BIONECT® Cream is a white, viscous cream. BIONECT® Gel is a clear, colorless gel. BIONECT® Spray is a clear, colorless, aqueous solution that is delivered as a spray. BIONECT® Foam is a soft white foam, which, at rest, takes the form of a clear, pale yellow liquid. The principal component is the sodium salt of hyaluronic acid (0.2%). The sodium hyaluronate (Hyalastine®) is derived from a natural fermentation process. Hyaluronic acid is a biological polysaccharide (glycosaminoglycan) and is a major component of the extracellular matrix of connective tissues.

**Indications:** BIONECT® is indicated for the dressing and management of partial to full thickness dermal ulcers (pressure sores, venous stasis ulcers, arterial ulcers, diabetic ulcers), wounds including cuts, abrasions, donor sites, and post-operative incisions, irritations of the skin, and first and second degree burns. The dressing is intended to cover a wound or burn on a patient's skin, and protect against abrasion, friction, and desiccation.

**Directions:** The wounds or ulcers should be cleaned and disinfected prior to treatment. In the event of long-standing ulcers, it may be advisable to clean and/or to debride the wound by surgical or enzymatic means, prior to treatment. Apply a thin layer of BIONECT® without extensive rubbing onto the wound surface, two or three times per day. Cover the lesion area with a sterile gauze pad and, if necessary, with an elastic or compressive bandage.

**Warnings:** If condition worsens, consult your physician immediately. Keep this product out of the reach of children. The prolonged use of the product may give rise to sensitization phenomena. Should this happen, discontinue the treatment and follow a suitable therapy. Do not use the product after the expiration date reported on the package or if the package is damaged.

### Ingredients:

BIONECT® Cream - Hyaluronic acid sodium salt, polyethyleneglycol 400 monostearate, decyl ester of oleic acid, emulsifying wax, glycerol, sorbitol solution 70%, sodium dehydroacetate, methylparaben, propylparaben, fragrance, purified water.

BIONECT® Gel - Hyaluronic acid sodium salt, sorbitol solution 70%, sodium dehydroacetate, methylparaben, propylparaben, carbomer 980, sodium hydroxide, purified water.

BIONECT® Spray - Hyaluronic acid sodium salt, sodium chloride, methylparaben, propylparaben, sodium dehydroacetate, purified water.

BIONECT® Foam - Hyaluronic acid sodium salt, purified water, butane, propylene glycol, polysorbate 80, polyvinylpyrrolidone,

propane, methylparaben, sodium dehydroacetate, propylparaben.

**Contraindications:** Do not administer to patients with known hypersensitivity to this product.

**Interactions:** Do not use concomitantly with disinfectants containing quaternary ammonium salts because hyaluronic acid can precipitate in their presence. The concomitant topical treatment of wounds with antibiotics or other local agents has never given rise to interactions or incompatibilities with BIONECT®.

**Precautions:** Avoid direct contact of container with the affected area. Each container of BIONECT® should be used by one patient only in order to reduce the risk of cross infection.

**WARNING:** BIONECT® Foam - contents under pressure, do not puncture or incinerate. Do not store at temperatures above 120°F. Protect from freezing. Keep out of reach of children.

**Adverse Reactions:** All suspected adverse reactions occurring during the treatment with BIONECT® should be reported to your doctor.

### How Supplied

BIONECT® Cream/Gel/Spray/Foam is supplied in a:

25g Cream - 68712-007-02

50g Cream (2 tubes of 25g) - 68712-007-03

100g Cream - 68712-007-04

30g Gel - 68712-008-02

60g Gel (2 tubes of 30g) - 68712-008-03

100g Gel - 68712-008-04

20 ml Spray - 68712-009-01

113.4g Foam - 68712-046-01

**Storage:** Please store BIONECT® at room temperature. BIONECT® Cream may be stored for up to 24 months, BIONECT® Gel may be stored for up to 24 months, BIONECT® Spray may be stored for up to 36 months, BIONECT® Foam may be stored for up to 24 months under these conditions. Store at room temperature below 86°F (30°C).

Manufactured by: Fidia Farmaceutici S.p.A. - Italy



Distributed by: Innocutis Holdings, LLC

Charleston, SC 29401

1-800-499-4468 • www.Bionect.com

U.S. Pat. No.: 5,925,626

Bionect and Hyalastine are registered trademarks of Fidia Farmaceutici S.p.A.



www.InnocutisCares.org

June 2014